

M99



Arterial Puncture Training Wrist

An essential simulator to increase confidence and competence

Arterial blood sampling and cannulation is a common yet delicate procedure.

With the innovative design of the arterial puncture simulator, a true-to-life experience can be simulated to ensure correct techniques and boost proficiency.

Product Supervision

Dr. Hiroshi Koyama,
Medical Director, General Internal Medicine
National Hospital Organization Kyoto Medical Center



FEATURES

- 1 | Palpable arterial pulse
- 2 | Realistic needle resistance against tissue and artery wall
- 3 | Natural counter-flow
- 4 | Radial arterial line placement can be simulated
- 5 | No puncture trace remains on the injection site
- 6 | Leak-free connections

SKILLS


- | Radial arterial puncture
- | Artery blood sampling
- | Radial line placement



Radial Arterial Puncture



Radial Arterial Line Placement

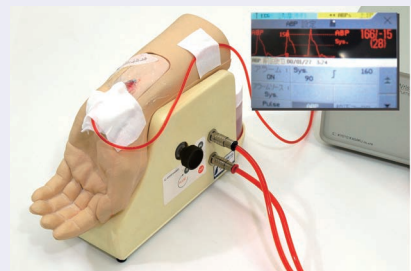

IABP Measurement
(Arterial pressure values are not simulated)

DESCRIPTIONS

For product codes of the alternate skin tone versions of this model, please refer to www.kyotokagaku.com.

SPECIFICATIONS

Size: W11 x D30 x H17 cm / W4.3 x D11.8 x H6.7 inch (manikin size)
W20 x D16 x H13 cm / W7.9 x D6.3 x H5.1 inch (pump size)

Weight: 1.5kg/3.3lbs
Power: AC100-240V, 50/60 Hz

MATERIALS

Soft resin
Latex free

RECOMMENDED DEVICES

AGB Kit 22-25G

SET INCLUDES

- | | |
|---|---------------------|
| 1 wrist and hand model | 10 cotton swabs red |
| 1 circulation pump | 1 spoon |
| 2 sets of puncture skins (inner and outer skin) | 2 plastic jars |
| 4 artery tubes | 2 connection tubes |
| 1 silicon tape | 1 vinyl sheet |

- | |
|----------------------|
| 1 syringe |
| 1 storage case |
| 1 instruction manual |

REPLACEMENT PARTS

- | | |
|-----------|----------------------------|
| 11351-010 | 1 replacement skin set |
| 11351-030 | 4 replacement silicon tape |
| 11351-040 | 4 replacement artery tube |
| 11388-400 | 10 cotton swabs red |

Ultrasound Guided Arterial Puncture Training Pads

Enhanced Training with the M99: Master Ultrasound-Guided Radial Arterial Puncture

Boost your confidence and skill in arterial blood sampling



Ultrasound Puncture pads



Training with M99 Arterial Puncture Training Wrist

FEATURES

- 1 | Palpable pulsation of arterial pulse
- 2 | Realistic needle resistance against tissue and artery wall
- 3 | Natural counter-flow
- 4 | Radial arterial line placement can be simulated
- 5 | No puncture trace remains on the injection site
- 6 | Leak-free connections


SKILLS

- | Ultrasound-guided
 - radial artery puncture
 - artery blood sampling
 - radial line placement


IMAGES



Transverse Plane

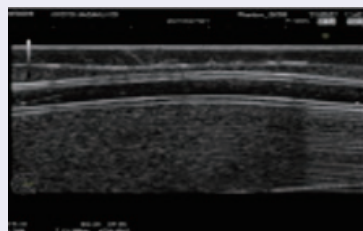

Longitudinal Plane

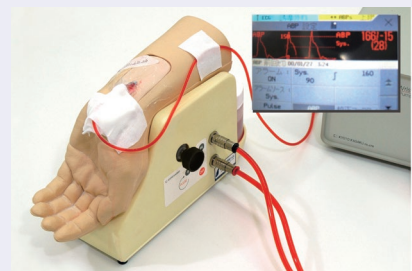

IABP Measurement
(Arterial pressure values are not simulated)

DESCRIPTIONS

REPLACEMENT PARTS

- 11351-110 2 ultrasound puncture pads
- 11351-120 2 ultrasound puncture inner pads
- 11351-130 2 ultrasound puncture outer pads
- 11388-400 10 cotton swabs red

MATERIALS

- Soft resin
- Latex free

RECOMMENDED DEVICES

- AGB Kit 22-25G

Next page : M99 Arterial Puncture Training Wrist

Ultrasound scanner and needles are not included.